## Instructions for use Calmita

## Manual de instrucciones Calmita

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## Instructions for use Calmita – Hospital feeding solution to support and protect breastfeeding

### Intended use

In combination with a Medela disposable hospital bottle, Calmita is intended to be used for oral feeding of human milk, fortified human milk or formula to preterm and term infants when breastfeeding is not possible.

Calmita allows infants to apply their natural feeding behaviour (sucking and swallowing, breathing and pausing), thereby supporting and protecting breastfeeding.

### Important Safeguards

#### Read all instructions for use For your child's safety and health WARNING!

- I Use under the guidance of a healthcare professional.
- I Always use this product under adult supervision.
- I This is a single-use product not intended to be reprocessed, e.g. do not wash. Reprocessing could cause loss of mechanical, chemical and/or biological characteristics. Reuse could cause cross contamination.
- I Single-use is defined as one feeding session.
- I Do not use when package is opened or damaged prior to first use.
- I Inspect before use. Throw away if there are signs of damage or weakness.
- I Only touch locking ring of teat in order to avoid contaminating the nipple.
- I Never use feeding teats as a soother/pacifier.
- I Continuous and prolonged sucking of fluids can cause tooth decay.
- I Always check the temperature of food before feeding it to your baby (risk of burning).
- I Do not warm milk in a microwave oven as this may cause uneven heating and could scald your baby.
- I Use Medela original accessories only to ensure proper function.

#### SAVE THIS INSTRUCTION MANUAL

### Application

#### How to use

- I Open packaging.
- I Screw Calmita to the bottle firmly, but do not over tighten to ensure proper function.
- I Start feeding session.

Note: Use packaging as hygienic protection to cover Calmita when not actively feeding.

#### Product types

There are two types of Calmita available to support and protect breastfeeding when direct breastfeeding is not possible:

##### *Calmita Starter (white)*

- I Low vacuum level to release content.
- I Suitable for preterm infants in transition from enteral to oral feeding, late preterm infants and term infants with a weak suck.

##### *Calmita Advanced (yellow)*

- I Moderate vacuum level to release content.
- I Suitable for preterm, late preterm and healthy term infants who are able to create the required vacuum level.

Calmita can be used with the following fluids:

- I Human milk
- I Fortified human milk
- I Liquid formula
- I Well dissolved powdered formula

### Cleaning and Hygiene

Calmita does not need cleaning prior to use as it is Ready-to-Use.

A new Calmita is required for each feeding session.

Disposal: According to local regulations

Material: Polypropylene (hard part), thermoplastic elastomer (soft part)

Distributed by:  
Distribuida por:  
Distribuée par:

Medela Inc.  
McHenry, IL  
Phone: (800) 435-8316

Medela AG  
Lättichstrasse 4b  
6341 Baar, Switzerland  
www.medela.com

 Manufacturer / Fabricante /  
Fabricant

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