Pump In Style® Battery Pack Instructions

IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS

DANGER: To reduce the risk of electrocution:

1. Always unplug electrical products immediately after using.
2. Do not use while bathing.
3. Do not place or store product where it can fall or be pulled into a tub or sink.
4. Do not place in or drop into water or other liquid.
5. Do not reach for a product if it has fallen into water. Unplug immediately.


WARNING: To reduce the risk of burns, electrocution, fire or injury to persons:

1. A product should never be left unattended when plugged into an electric outlet.
2. Close supervision is necessary when this product is used by, on or near children or invalids.
3. Use the product only for its intended use as described in these instructions. Do not use attachments not recommended by the manufacturer.
4. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
5. Keep the cord away from heated surfaces.
6. Never use while sleeping or drowsy.
7. Do not use outdoors, or operate where aerosol (spray) products are being used or where oxygen is being administered.
8. Only use the Medela® Pump In Style Battery Pack with the Medela Pump In Style breastpumps.
9. Always inspect power adaptor and battery pack wires prior to use for damage or exposed wire. If damage is found, immediately discontinue use of power adaptor or battery pack and call Medela Customer Service at 1-800-435-8316.

The Pump In Style Battery Pack

The Pump In Style Battery Pack is intended for use with Pump In Style breastpumps (manufactured in January 2008 and later) ONLY. Battery Pack uses 8 AA alkaline batteries, or, for more consistent pump performance, use rechargeable batteries. It is recommended that batteries be replaced or recharged between the 10th and 12th pumping session.

Battery Insertion

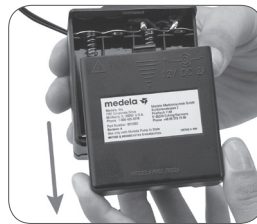Replace all batteries at the same time.

Slide off both battery covers. Four (4) batteries must be on each side of battery pack.

Place the NEGATIVE (-) end of the battery against the front of the spring by pushing the battery in and down in one motion until the battery snaps into place.

The POSITIVE (+) end of the battery should be near the positive sign on the battery pack.

Replace battery cover.

- Remove the batteries if the Battery Pack is not going to be used for an extended period of time.

- If breastpump does not turn on, make sure the batteries have been inserted properly. If this does not solve the problem, contact Medela Customer Service at 1-800-435-8316.

- Each set of batteries will provide approximately 2 hours of pumping time.

Battery Removal

Slide off the battery covers. Insert a BLUNT, NON-METALLIC tool into the removal slot on either side of the Battery Pack. Gently pry the battery out. Replace the battery covers.

The Battery Pack was designed for safe use while stored in its pouch. The pouch fits inside the main bag or motor bag, along with the transformer.

SAVE THESE INSTRUCTIONS

medela 

Medela, Inc., 1101 Corporate Drive, McHenry, IL 60050
Phone: (800) 435-8316 or (815) 363-1166 Fax: (815) 363-1246
Email: customer.service@medela.com www.medela.com

Medela and Pump In Style are registered trademarks of Medela Holding AG.

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Printed in the USA 1907518 D 0613 ©2013 Medela, Inc.
Medela et Pump In Style sont des marques déposées de Medela Holdings AG.
Imprimé aux États-Unis 1907518 D 0613 ©2013 Medela, Inc.