

# Colostrum Collector

Protect Every Drop



Colostrum is a powerful therapy for the NICU infant and can lead to improved patient outcomes.<sup>1</sup> Every drop of colostrum is valuable and should be protected from loss. The Colostrum Collector is easy to use and seamlessly connects to the Symphony® Breast Pump, without impacting pump performance. It catches all expressed drops of colostrum without getting lost in the breast pump parts, making it more effective than most traditional methods.

## Inspired by insights from nurses

- **Simple and easy to use:** The Colostrum Collector is easy to use and seamlessly connects to the Symphony® Breast Pump, without impacting pump performance.
- **Minimizes loss of colostrum:** The Colostrum Collector minimizes loss of colostrum by catching it prior to reaching the pump parts; ensuring baby can get every drop.
- **Effective:** 84% of nurses find the Colostrum Collector to be more effective at collecting colostrum than pumping without it, allowing you to collect colostrum with confidence.<sup>2</sup>
- **Easy to transport:** The Colostrum Collector’s snap on lid makes it easier to transport and protects from contamination, allowing you to focus on patient care.
- **Increases mom’s sense of accomplishment:** The Colostrum Collector is perfect for holding the small amounts of colostrum produced in those first few days. 78% of nurses felt mom had a greater sense of accomplishment using the Colostrum Collector.<sup>2</sup>
- **Single Use:** Supports HMBANA best practice recommendation for NICU patients, which recommends using a new container for each pumping session.<sup>3</sup>

## How does the Colostrum Collector work?\*

- 1 Colostrum Collector is attached to the Symphony® kit, between the breast shield and the connector.
- 2 Mom uses the Symphony breast pump to initiate breast milk production.
- 3 Colostrum is collected in the Colostrum Collector before reaching the rest of the pump parts, minimizing loss.  
**Tip:** In the event mom comes to volume sooner than expected and milk volumes exceed the 5 mL capacity; it will overflow into the container below.
- 4 After pumping, remove the Colostrum Collector from the Symphony kit keeping the breast shield on and set on a flat surface. Remove the breast shield and snap the lid onto the Colostrum Collector



## Product Specifications:

Product Name	Item Number	Description	Unit (Each Information)		Shipper Information		
			Dimensions (H x W x D Inches)	Weight (lbs)	Qty/Case	Dimensions (H x W x D Inches)	Weight (lbs)
Colostrum Collector 2-Pack	101044659	Includes 2 Colostrum Collectors and lids	2 x 5.5 x 4	0.06	40 packs of 2	16 x 10.875 x 9.25	2.4
Symphony Breast Pump Kit with Colostrum Collector	101044744	Includes 1 Symphony Initiation Kit and 4 Colostrum Collectors and lids	4 x 9.5 x 11.5	0.5	10 kits	22 x 12 x 19	5

Content appearance may vary.

<sup>1</sup>Meier PP et al. J Pediatr. 2017;180:15-21. <sup>2</sup> Medela 2022 Colostrum Collector Evaluation Survey <sup>3</sup> Jones, F. Best Practice for Expressing, Storing, and Handling Human Milk in Hospitals, Homes, and Child Care Settings. 4th ed Fort Worth, TX: Human Milk Banking Association of North America, Inc. (2019), pg 59

\*Refer to Instructions for Use for complete instructions.