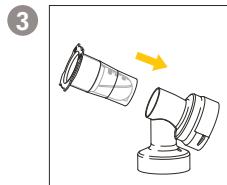
## Assemblage :

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**Content appearance may vary.** Medela wordmark and logo is registered in the U.S. Patent and Trademark Office and the Canadian Intellectual Property Office.

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## Product Description | Descripción del producto | Description du produit :

The sterile, single-use 6 mL Colostrum Collector is a container with lid. The Colostrum Collector is designed to fit seamlessly between the Symphony® breast pump connector and breast shield and is for use with the Symphony breast pump.

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### ⚠ WARNING

Can lead to serious injury or death.

- Do not use if the packaging is open or damaged prior to use.
- The packaging should be inspected for holes, tears, broken seals, etc.
- This is a single-use product not intended to be reused or reprocessed. Reprocessing could cause loss of mechanical, chemical and/or biological characteristics. Reuse could cause cross-contamination.
- Use a new disposable Colostrum Collector for each pumping session.

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### CAUTION

- Can lead to minor injury.
- Use Medela Symphony breast pump kit only.

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
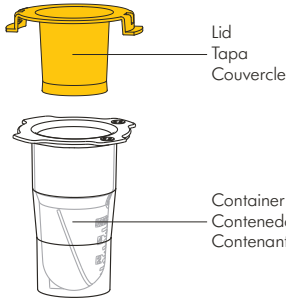
### EN

### Assembly:

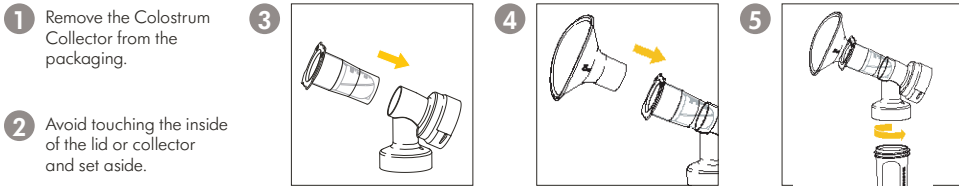The Colostrum Collector is sterile and does not need to be cleaned prior to use.

Attach the bottom of the Colostrum Collector to the Symphony breast pump kit connector. Orient Colostrum Collector with the two raised marks (smiley faces) facing up.

Attach the breast shield to the top of the Colostrum Collector.

Assemble the remaining breast pump kit as directed in the Symphony breast pump kit instructions.

**Note:** Orientation is important to ensure the vent hole is facing up. Orienting any other way may result in colostrum spilling from vent hole and into the connector.

### Disassembly:

- Gently remove Colostrum Collector from connector and leave the breast shield attached.
- Place the Colostrum Collector on a flat surface.
- Remove the breast shield.
- Secure the lid to the Colostrum Collector.
- Store colostrum according to hospital guidelines.
- After administering colostrum per hospital guidelines, dispose of or recycle\* the Colostrum Collector and lid.

**Warranty:** Limited 90-day warranty. For complete warranty information visit the product page at [Medela.us](http://Medela.us) or [Medela.ca](http://Medela.ca).

**Material:** Polypropylene

**Disposal:** According to local regulations or hospital protocol.

### Meaning of Symbols:

Warning sign	Caution sign	Indicates the manufacturer	Indicates the date of manufacture
Global trade item number	Indicates the manufacturer's product number	Indicates the manufacturer's batch code or lot	Indicates expiration date
Indicates Ethylene Oxide EO Sterilization	Indicates do not use if the package is damaged	Indicates open here	Indicates single use only
Read and follow the instructions for use	Recyclable		

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1549914 A 0422 © 2022 Medela 101044788